# normomar<sup>®</sup> spray

NMR-MAR100/4.3

## Agua Marina Higiene nasal diaria

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# normomar<sup>®</sup> spray

## Sea Water Daily nasal hygiene

Nasal cleaning is key to a good breathing.

When the nose is too dry or congested it cannot filter air properly.

Thanks to its **droplet technology and the benefits of marine trace elements, Normomar provides a gently cleaning and decongests noses.**

### INDICATIONS:

Normomar is a healthcare product indicated for **newborn babies, children and adults:**

#### · Daily nasal hygiene:

· It induces the need to blow the nose and makes it easier,

· It restores natural hydration of the nose (it fights the effects of air conditioning and heating),

· It prepares and enables the action of specific nasal treatments and improves their effectiveness.

· **It removes nasal mucosa in blocked and congested noses.**

### USAGE INSTRUCTIONS:

Normomar may be used in all positions.

The unique nozzle is specifically designed to fit the nose of newborn babies and adults:

1. Gently insert the nozzle into one nostril.
2. Prime the spray by briefly depressing the nozzle. If your nose is congested or blocked, depress the nozzle for some seconds.
3. Allow the excess solution to come out, as it will in turn carry the mucus.
4. Repeat the process in the other nostril.
5. Clean the nozzle after application with soapy water, rinse and dry.

**Use this product as nasal irrigation where necessary.**

Normomar is a natural product free of preservative agents and may be used for prolonged periods of time.

Do NOT use after the expiry date stated in the package and label.

### COMPOSITION:

Normomar is 100% natural; it is an isotonic\* saline solution providing the well-known benefits of sea water mineral salts and trace elements: copper, iron, manganese, selenium, zinc, etc.

\*Isotonic solution: a solution containing the same concentration of salt as a human body.

Sea water: 30 mL/100 mL approx.

Purified water q.s.: 100 mL.

The container has been pressurized with a propellant gas (air). Thanks to its breakthrough bag on valve technology, the solution is not in direct contact with the propellant gas (air) in order to preserve the benefits of sea water. Using this product by more than one person may lead to spreading infections. Pressurised containers must be protected from sunlight. Do NOT expose to temperatures over 50 °C. Do NOT pierce or burn, even after use. Do NOT use if package or product are damaged.

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Naveh Pharma Ltd
Obelis, S.A.
LABORATORIOS NORMON, S.A.